





# Instructions:

# 4Closure™

SURGICAL FASCIA CLOSURE SYSTEM

## Step 1

Insert **4Closure™** device through cannula, or if cannula has been removed, through trocar site. Remove cannula (if not already removed). Attach suture to suture passer. Insert suture passer through hole in **4Closure™** guide rod. Adjust depth of guide rod so that the needle exits the guide hole below the level of the skin. Make sure that the suture passer catches approximately one cm of fascia. Observe entrance of needle into abdominal or pelvic cavity to avoid injury to surrounding structures. Open suture passer and tug on the suture to release it. Remove suture passer instrument.

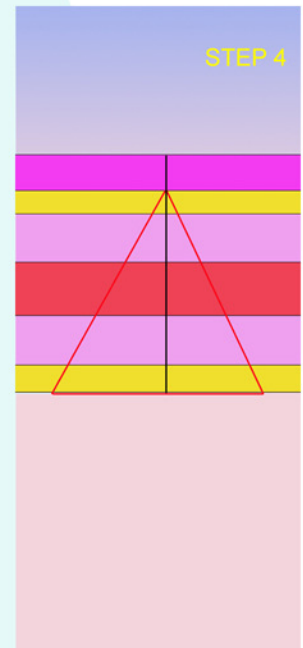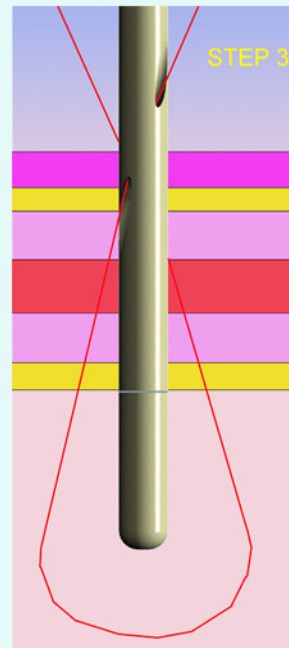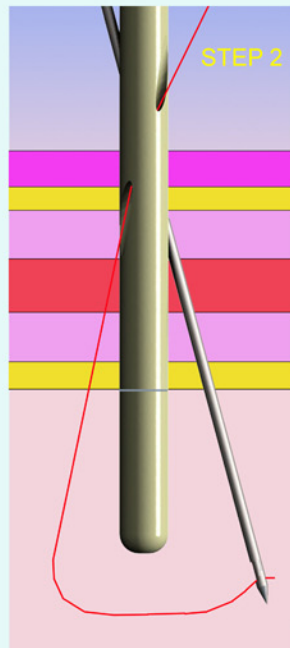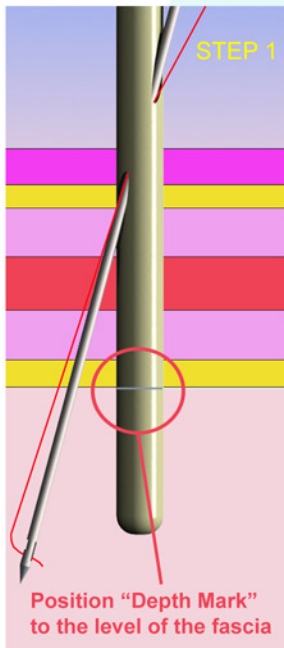

**NOTE:**  
Do not rotate the guide rod after it is placed.

**Step 2**  
Re-insert suture passer through opposite guide hole.

**Step 3**  
Engage suture into needle and remove **4Closure™** guide rod. Apply tension to suture and insert index finger into wound to assure proper placement of suture. Observe wound from inside to assure closure and hemostasis.

**Step 4**  
Tie suture. Re-insert finger into wound to assure proper fascial closure.



The 250mm long **4Closure™** guide rod contains one 15 degree guide hole on each side for optimal closure of the fascia.

[www.allezmedical.com](http://www.allezmedical.com)